


AIRSTRIP®

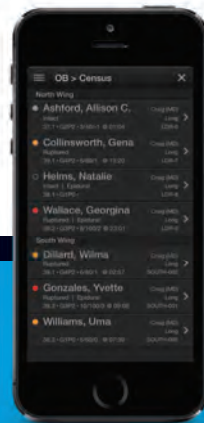

POCKET GUIDE  
AIRSTRIP ONE®  
for obstetrics

## **INTRODUCTION**

**AirStrip ONE for Obstetrics delivers visibility into monitored patient data in the following categories:**

- Real-time monitor viewing
- Patient Demographics

**Supported data include:**

- Waveforms
- Dilatation Status and Cervical Exams
- Annotations
- Maternal Vitals

**Launching and using AirStrip ONE:**

1. Tap the AirStrip ONE app
2. Tap the site's name
3. Log in with your site credentials


**To get started or get  
help, call 877-258-5869  
[www.airstrip.com](http://www.airstrip.com)**

**Important Note: Users must  
contact AirStrip at (877) 258-5869  
for re-installation of AirStrip  
ONE if they acquire a new mobile  
device.**

**For complete instructions please  
see the user guide.**

## ACCESSING PATIENT DATA

Locate a patient using SEARCH, or from the default patient list.

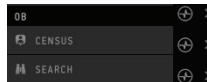

## PATIENT CENSUS SCREEN

The Patient Census Screen displays a list of patients with various fields and actions. Annotations include:

- Tap the Navigation Menu button to view the Navigation List and to Search for a patient.
- Tap the X to close the window.
- Doctor Last Name, Nurse Last Name, Patient Location. Note: Display of the doctor and nurse's name is configurable by Site.
- Tap to view the Patient Summary screen.
- Scroll to view additional patients.

Unit Name	Dilation Status	Estimated Gestational Age (EGA)/Effacement/Station and Time of last exam	Membrane Status	Doctor Last Name	Nurse Last Name	Patient Location
South Wing	Dillard, Wilma					
	Gonzales, Yvette					
	Williams, Uma					
West Wing	Babbitt, Cindy					
	Harris, Wendy					
North Wing	Ashford, Allison C.					
	Collinsworth, Gena					

## DILATION STATUS

Label	Patient Name	Status	Provider	Location
Patients that are 1-4.9 centimeters dilated	Ashford, Allison C.	Intact	Craig (MD)	Long LDR-6
Patients that are 5-7.9 centimeters dilated	Collinsworth, Gena	Ruptured	Craig (MD)	Long LDR-7
Patients not yet documented	Helms, Natalie	Intact   Epidural	Craig (MD)	Long LDR-8
Patients that are 8-10 centimeters dilated	Wallace, Georgina	Ruptured   Epidural	Craig (MD)	Long LDR-8

## PATIENT SUMMARY SCREEN

The header displays the patient name, age, gender, hospital name, unit, and bed number.

Tap the **Action Overflow** button to share the screen with another AirStrip ONE user, toggle strip colors or change column width.

Tap the **X** to close the screen.

Tap the **caret** button to show/hide the waveforms.

Tap the **Live (real-time)** button to return to live (real-time) view.

Tap the **Annotation** icon to view an annotation.

**Time Ago** indicator displays when accessing historical data.

Dilation and Membrane status.

EGA/G, P/Dilatation/Effacement/Station.

Pinch/zoom to zoom in/out on strips. Note the change in time and % of the strip chart displayed.

Swipe right view history for up to 24 hours.

Drag the screen divider up to view tiles.

Tap a tile to view more patient data.

CERVICAL EXAM	W/DOCN	W/HL	
08:00	3 / 100 / 2	1001	0 HL
08:30	8 / 100 / 2	1002	1 HL
09:00	8 / 100 / 2	1003	2 HL
09:30	8 / 100 / 1	1004	2 HL
10:00	7 / 90 / 1	1005	1 HL
10:30	7 / 90 / 1	1006	1 HL
10:12	7 / 90 / 0		

## MATERNAL VITAL SIGNS SUMMARY TILE

ASSESSMENTS	VITALS	TRENDS
08:00 DR KING AND MED...	177	101.8 - 102.2 - 102.8
08:00 DR KING HERE (1)	90	80 - 88 - 90
08:00 NICU TEAM HERE...	230	110 - 118 - 111
08:00 URINE CALLED AW...	105	87 - 79 - 72
08:00 DR KING PAGED N...	86	— - 84
08:00 DR KING BRING N...	27 SpO2	88 - 89 - 88
08:00 Davidson, B. Etson...		

High - Low range and most recent Vitals for up to 23 hours for:

- T (Temperature)
- HR (Heart Rate)
- SBP (Systolic Blood Pressure)
- DBP (Diastolic Blood Pressure)
- RR (Respiration Rate)
- O2% (Oxygen Saturation)

Note: Summary tiles display a summary of the patient data available on the expanded tile. Once a summary tile is tapped, the expanded tile displays.

AIRSTRIP®

www.airstrip.com

# AIRSTRIP®

335 E. Sonterra Blvd., Suite 200  
San Antonio, TX 78258  
+1 210 805 0444  
[www.airstrip.com](http://www.airstrip.com)

© 2017, AirStrip Technologies. All rights reserved. Trademarks not belonging to AirStrip Technologies are the property of their respective companies. Screen images contained within are for illustrative purposes. Actual screens may vary by implementation, site and user.  
AirStrip ONE for obstetrics iOS pocket guide V 1.5 2017-10-31



EMERGO EUROPE  
Pharosweg 10  
2514 AF The Hague  
The Netherlands